

5IF "*3OJWB 3FTQJSBUPSZ (BT)VNJEJGJFS
 DPNQMFNFOUT UIF 8*-"NFE QPSUGPMJP GPS
 SFTQJSBUPSZ DBSF TPMVUJJPOT XJUI TQFDJBM GPDVT PO
 OPOJOWBTJWF WFOUJMBUJPO 5IF "*3OJWB QSPWJEFT
 PQUJNBM IFBUFE BOE IVNJEJGJFE HBT TVQQPSUJOH
 QBUJFOU DPNQMJBODF UP /*7 USFBUNFOU JO BO
 FBTZ BOE TBGF NBOOFS VMUJNBUFFMZ FOIBODJOH
 QBUJFOU DPNGPSU

*U JT BEKVTUBCMF CFUXFFO UISFF IVNJEJGJDBUJPO
 NPEFT NBLJOH JU UIF QFSGFDU DIPJDF GPS B
 XJEF SBOHF PG /*7 UIFSBQJFT TUBSUJOH GSPN
 #J-FWFM UIFSBQJFT JO UIF IPNF JOUP IJHI MFWFMT PG
 WFOUJMBUPSZ TVQQPSU JO UIF IPTQJUBM

5IF "*3OJWB DPNFT XJUI 8*-"NFE T OFX "54 "NCJFOU 5FNQFSBUVS
 WBSJPVT BNCJFOU DPOEJUJJPOT XIJDI DBO CF GPVOE JO IPNF BO
 DPME ESZ BOE BOIZESPVT HBT EFMJWFSFE UP UIF QBUJFOU WJB
 DBO SFTVMU JO IFBU BOE NPJTUVSF MPTT BOE ESZJOH XIJDI DBO
 CSFBUIJOH

Part Number	Description	Box / Qty.
	"*3OJWB #BTJD 6OJU 7	
)FBUJOH XJSF BEBQUFS J	
	5FNQFSBUVSF QSPCF DN	
Accessories	\$ "' 6OJWFSTBM)VNJEJGZJOH \$IBNCFS	
	4JOHMF MJNC TZTUFN VOIFBUFE VOJWFSTBM BVUPG. DN NN • \$POOFDUJOH UVCF DN	
Consumables	8*-"RVB 4UFSJMF 8BUFS NM	

OFFET UP CF PSEFSFE TFQBSBUFMZ

the new standard for humidification in NIV

AIRniva provides a complete respiratory humidification system in combination with its specially designed humidifier chambers and breathing tube systems. It is suitable for non-invasive applications such as maskventilation or O₂-Therapy.

User-friendly

- Easy application with two symbol buttons
- Automatic function mode detection at start
- 3 function modes
- Humidification control display

Efficient

- Suitable for all common non-invasive ventilators
- Ready for immediate use
- Extended accessory range

Safe

- Easy alarm management

Quality

- Protection Class II for clinical use
- Less maintenance, no hidden costs
- "Made in Germany"



Smart performance

The AIRniva features three humidification modes:

- **NHW – Non Heated Wire Operation**
Without Heater Wire, without temperature probe
- **HWC – Heated Wire Calculated**
With Heater Wire, without temperature probe
- **HWT – Heated Wire Temperature probe**
With Heater Wire, with temperature probe



AIRniva

